


SUPERTOUCH® How To: Safely Remove Contaminated Gloves

1.

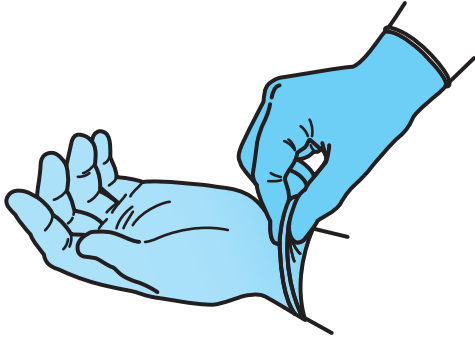

At the wrist, pinch and fold the cuff of the glove towards the fingertips.

2.



Pull the glove away from your body. Make sure the glove turns inside out as you do.

3.




Continue to pull the glove. Once removed, hold it with your gloved hand.

4.



Slide your finger under the cuff of the other glove and **repeat Step 2**.

5.

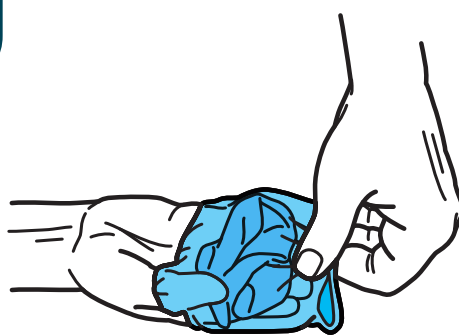

Encase the first glove into the second one as you remove it.

6.



Dispose of the gloves. Wash your hands before touching any other surfaces.

www.supertouch.com
sales@supertouch.com
0345 130 9922

Stay Clean. Stay Safe. Stay Alert.